# 9200-R

## Tub Shower Board

### Assembly Instructions

#### Installation Instructions

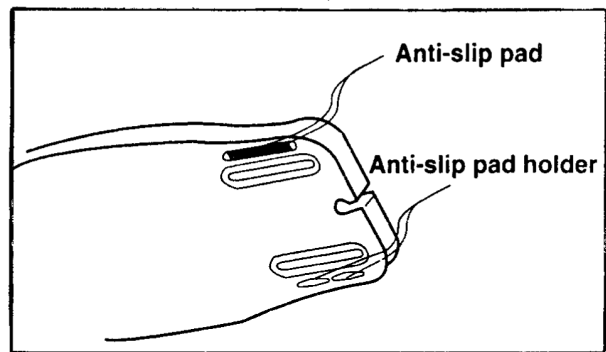
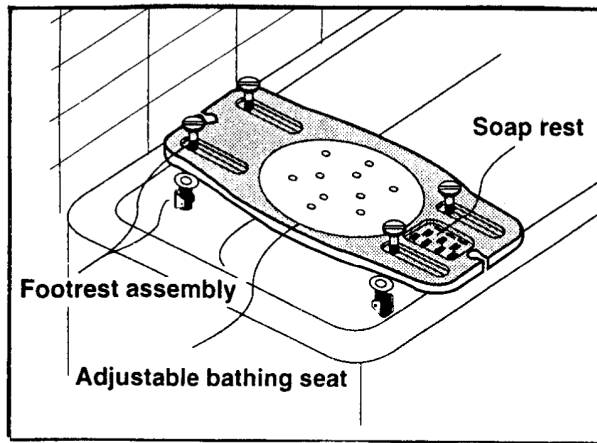
1. Unscrew knobs from foot assembly
2. Place knob through the slots so knob handle is facing upward
3. Install anti-slip rubber pad into anti-slip pad insert located on the bottom of the bath board next to the open slots
4. Loosely assemble the four foot rest assemblies to the shower board through the slots by placing the knobs on top of the board and the rubber stops underneath, and then screwing the two pieces together
5. Be certain inner walls of tub are free from dirt, grease, or soap build-up
6. Place tub board on top of tub with soap dish closest to wall, away from the side of entry
7. Position the rubber stops of each foot assembly against the inner wall of the tub
8. Tighten each knob from the top while pushing rubber stops against the inner tub wall  
Check knobs periodically to ensure rubber foot assemblies are securely tightened to tub walls

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### Safety and Care Instructions

- Do not use red safety handle as weight support when sitting or standing
- Check red handle periodically to ensure grab bar is securely tightened to bath seat
- Extreme caution should be used on wet surfaces
- Persons with limited physical/mental abilities should be assisted during bathroom activities
- Clean with soap and water

**Manufacturer bears no responsibility for improper use and/or installation**

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## **IMPORTANT**

**READ THESE DIRECTIONS CAREFULLY BEFORE USE.  
FAILURE TO DO SO MAY RESULT IN SERIOUS INJURY.  
WEIGHT CAPACITY NOT TO EXCEED 250 LBS.**

---

Nova stands firm on our commitment to providing the highest quality products, as well as exceptional service. This product was built to precise standards and thoroughly inspected prior to shipment. This warranty represents our confidence in the materials and workmanship of our product.

Nova extends this warranty only to the original purchaser of this product. The warranty does not extend to any subsequent purchaser or owner. The warranty is void upon any sale or transfer of ownership or use by other person. Nova warrants its products are free from defects in material and workmanship under NORMAL use for the period commencing upon the date of purchase and continuing for the following specified period of time after that date:

**Plastic Frame: Limited Lifetime Warranty**  
**Rubber Clamps and Plastic Hardware: 1 Year Warranty**

This warranty does not cover problems due to user negligence, misuse of the product, failure to adhere to product instructions, or normal wear and tear. Items that have been subject to negligent abuse, modified without prior written authorization are not covered within this warranty.

For warranty service, please contact the dealer from whom you purchased your Nova product. In the event that you do not receive satisfactory service, please call us directly at the number below. Be prepared to indicate the nature of the defect, the product serial number, and the name and location of where you purchased the product. You will also be required to provide a receipt of your purchase. **DO NOT** return any products to our office without prior consent. Repair or replacement is the only remedy under this limited warranty. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of product.

THIS WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES, IMPLIED WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, NOVA SHALL NOT BE LIABLE FOR ANY SEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.