CAUTION:

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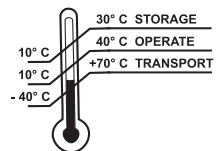
Read Instructions Before Use

Non Sterile

STORAGE
10 to 75 % non-condensing
OPERATE
30 to 75 % non-condensing
TRANSPORT
10 to 100 % including condensation

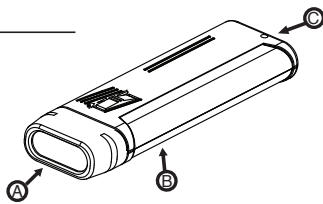
Bovie Medical Corporation
5115 Ulmerton Road
Clearwater, Florida 33760 USA

EC REP

Emergo Europe
Molenstraat 15
2513 BH, The Hague
The Netherlands

CAUTION:
Avoid Prolonged Direct Visual Exposure.

SPECIFICATIONS:

- Fluorescent Lamp (replaceable (REF UV59B)) – approx. 365 nm
- LED (light emitting diode) Lamp (non-replaceable)
- Power – 6V (replaceable 4 AA Batteries)

Medical Purpose/ Indication

The UV Woods Light is a hand held, lightweight, battery operated, portable light with optional ultraviolet lighting.

Operating Principle

Used in diagnostic visualization and semen detection.

Patient Population

No restriction

Site of Use and Site Conditions

Site of use – Skin
Site conditions – No restriction

Intended User Profile

This device is intended to be used by trained healthcare professionals.

Intended Conditions for Use

- Environment**
- Limited functionality in wet conditions
 - Clear and unobstructed view of point of use

Frequency of use

No restrictions up to life of battery or lamp

Disposal

- Fluorescent lamp – Dispose of lamp following fluorescent lamp disposal guidance
- Biohazard – Dispose of UV Light according to facility procedures

INSTRUCTIONS:

Remove protective plastic tape to access Woods Light switch.

OFF/No Light

Fluorescent Lamp On

LED Lamp On

FLUORESCENT LAMP REPLACEMENT INSTRUCTIONS

Squeeze both ends of the (B) lamp cover, and slide it out of position. Remove the fluorescent bulb by rotating it 90 degrees and lifting it out. Replace the bulb. Insert and snap the (B) lamp cover back into place.

NOTE: The LED Lamp (A) is non-replaceable.

BATTERY REPLACEMENT INSTRUCTIONS

Slide battery cover to open (C). Remove all 4 batteries. Observing polarity, replace batteries.

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