1

## ENGLISH

**BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.**

### INTENDED USER PROFILE:

The intended user should be a licensed medical professional, the patient or the patient's caregiver.  
The user or caregiver should be able to read, understand and be physically capable of performing all the directions, warnings and cautions provided in the information for use.

### INTENDED USE/INDICATIONS:

The Procure Hip Abduction Pillow is designed to help position the hips away from the centre of the body (abduction). It may be suitable for post-operative use to maintain hip abduction.

### PERFORMANCE CHARACTERISTICS:

Providing immobilization or controlled movement of the limb or body segment.

### CONTRAINDICATIONS:

If you have skin disorders/injuries in the relevant part of the body, the product should be used only after consultation with a medical professional.

### WARNINGS AND PRECAUTIONS:

- If you experience any pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your medical professional immediately.
- Do not use over open wounds.
- Do not use this device if it was damaged and/or package has been opened.

**NOTE:** Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

### APPLICATION INFORMATION:

- 1) Place pillow between patient's legs with narrow end of pillow toward body. Slide pillow upward until the prescribed angle of the hips is obtained. Legs should fit comfortably into the contours on the sides of the pillow.
- 2) Secure pillow to legs with straps, fastening the pressure-sensitive closures for a snug but comfortable fit.

### CLEANING INSTRUCTIONS:

Hand wash in warm water using mild soap. Rinse thoroughly. AIR DRY. Note: If not rinsed thoroughly, residual soap may cause irritation and deteriorate material.

### WARRANTY:

DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

### INTENDED FOR SINGLE PATIENT USE.

 NOT MADE WITH NATURAL RUBBER LATEX.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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©2020 DJO, LLC