


## Window Alarm

Reorder # 909222

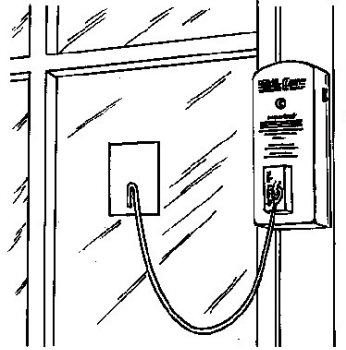

### Purpose

Skil-care's Window Alarm is intended to alert caregivers when a resident/patient opens a window. It is designed to be used on a window that is fully closed or one that is left slightly open for ventilation purposes.

### Warning

The Window Alarm is not intended for use on fire and emergency exit windows.

### Instructions

1. Install 9-volt battery. Battery compartment is on the back of the alarm unit.
2. Separate the Velcro® loop material strip from the back of the alarm unit. Peel the backing off Velcro® strip and attach it to the window frame.
3. Attach the alarm to the window frame by pressing the hook material on the alarm against the loop material on the window frame.
4. Separate the Velcro® strips attached to the magnet pull-cord. Peel the backing off the loop material and attach it to the windowpane. Press the hook material on the pull-cord to the loop material on the window.  
(See illustration above for proper positioning of alarm unit and window tab.)
5. Move the alarm switch to the HI or Low position. The alarm is now activated.  
The alarm is deactivated by moving the switch to the OFF (center) position.

### Important

Test the Window Alarm before each use. Move the switch to the HI position and remove the magnet. If alarm is weak or fails to sound, remove the battery and replace it with a fresh one. Retest. If alarm still fails to sounds, discontinue use.