# WelchAllyn®

## Rechargeable Battery Handles and Chargers

Before using this product, please read the warnings and instructions.

### General Use Instructions

### Maintenance

Like any other piece of electro-medical equipment, perform periodic electrical inspections by qualified personnel. Welch Allyn recommends that this equipment is inspected at least every six months, more often if used under adverse conditions.

### Instructions for Recycling

#### Nickel-Cadmium Batteries

Welch Allyn now employs the services of an agency which can disassemble and recycle all components of nickel-cadmium batteries so that nothing gets landfilled or incinerated. Should you wish to dispose of your expended Welch Allyn rechargeable battery via recycling (in the U.S. and Canada only), please send to:

Welch Allyn RECYCLE BATTERY 4341 State Street Road Skaneateles Falls, NY 13153-0220	Welch Allyn Canada RECYCLE BATTERY 160 Matheson Blvd. East Mississauga, Ontario CANADA L4Z 1V4
---	--

For locations outside the U.S. and Canada, consult your local recycling authority for correct disposal of batteries.

### Service:

For customers in North America, please return instruments requiring service to a Welch Allyn Technical Service Department listed below or to an authorized Welch Allyn Technical Service Department.

Technical Service Department  
Welch Allyn, Inc.  
4341 State Street Road  
Skaneateles Falls, NY 13153-0220  
Telephone: 1-800-535-6663  
Fax: 315-685-4653 (Technical Services)

In Canada:  
Telephone: 1-800-561-8797  
Fax: 1-905-890-0008

### Guarantee

Welch Allyn nickel-cadmium batteries are guaranteed by Welch Allyn for two years from date of manufacture (when used in Welch Allyn instruments only). Defective batteries are replaced on a pro rata basis should failure occur prior to expiration date on battery.

The Welch Allyn products listed on this document are guaranteed by Welch Allyn against all manufacturing defects. Welch Allyn will repair or replace, free of charge, any parts of its own manufacture proven to be defective through cause other than misuse, neglect, damage in shipment, or normal wear.

Welch Allyn warrants that the products will perform to original specifications during the two years from the date of manufacture. This warranty does not apply to the use of the 72200 or 72300 battery in other products.

### Environment

Transport/Storage: -20°C - +49°C, 95% R.H. Max, 500hPa - 1060hPa Altitude

Operating: 10°C - 40°C, 95% R.H. Max, 500hPa - 1060hPa Altitude

### Symbols:

Certified to: CAN/CSA C22.2 No. 601.1  
Conforms to: UL STD 2601-1

### IPX0

Equipment not protected against the ingress of water.

### Continuous Operation

NOTE: This product complies with current required standards for electromagnetic interference and should not present problems to other equipment or be affected by other devices. As a precaution, avoid using this device in close proximity to other equipment.

### Handles

### Safety and Warnings

- POSSIBLE EXPLOSION HAZARD. Equipment not suitable for use in the presence of a flammable anesthetic.
- Do not disassemble or modify the battery pack. There are no serviceable parts inside.
- Do not charge battery pack in any environment where the temperature may exceed 40°C (104°F) or fall below 0°C (32°F).
- For longest service, remove the handle and AC Charging Module from outlet when charging is complete. Do not leave AC Charging Module connected to power outlet without a handle attached.
- Do not sterilize the 710 Series Power Handle

#### FOR BEST PERFORMANCE:

- Do not expose battery pack to storage temperatures above 50°C (122°F) or below -20°C (-4°F).
- Do not discharge battery pack if the ambient temperature is above 60°C (140°F) or below -20°C (-4°F).

### Nickel Cadmium Handle

The 710 is a universal handle which accepts all Welch Allyn 3.5v instrument heads. Its adjustable rheostat control allows optimal intensity and battery life. The chrome-plated brass handles and metal hand connection provide durable, long life construction and the smooth, non-abrasive knurled finish insures a no-slip grip.

The 710 series also have a built in charger for easy, overnight charging of the long-lasting rechargeable nickel cadmium batteries. As an accessory, the converter ring is available in the 71000-C handle, which enables insertion of off-the-shelf C-cell batteries in place of the rechargeable batteries for continued operation without the need to wait for recharging.

#### Specifications:

Dimensions: 5.5" Long x 1.50" Diameter (max)  
Weight: 10 oz. (including battery)  
Input Voltage: 120V 60Hz 0.17A

#### Before using, follow these instructions:

1. fully charge battery (16 hours)
2. run battery completely down
3. fully recharge (16 hours)
4. run battery completely down
5. fully recharge (16 hours)
6. ready for use


### Operating Instructions:

1. Connect instrument head to the handle.
2. Depress On/Off Button on Rheostat Section and rotate the Light Intensity Control Ring clockwise (CW). To intensify light, continue to rotate until stop is reached.

To turn off, rotate counter clockwise (CCW). Button will click when full off. Turn fully off after each use to ensure maximum on-time from the battery.

### Charging Instructions for Rechargeable Handles with Self-contained Charger

1. Remove the instrument head and unscrew top cap from battery housing.
2. Plug into standard (110-120V/50/60 Hz) wall outlet. Charge for 16 continuous hours when new, or after a long period of activity.
3. For subsequent recharging, a single overnight charge is sufficient.

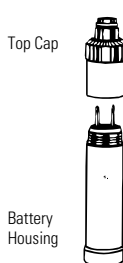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
### How to Replace Rechargeable Batteries

When a rechargeable battery will no longer accept a charge, replace as follows:

1. Unscrew bottom cap.
2. Shake out old battery.
3. Insert replacement battery "button" side up.

**CAUTION:** Insert battery fully until seated. For convertible handles, match polarity markings with those on instrument when inserting replacement battery.

4. Screw bottom cap back into place. Ensure the plastic insulator cap is securely attached to the bottom cap. If the insulator is missing or broken, contact Welch Allyn for a new bottom cap.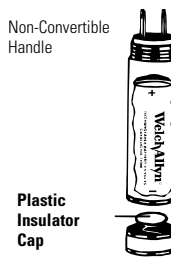


**For models 71000, 71010, 71020, 71050, 70700, 70710, 70720, and 70750 (non-convertible, rechargeable handles with self-contained charger), insure that plastic insulator cap is intact and securely seated on spring. If missing or damaged, install new cap before using. DO NOT INTERCHANGE BOTTOM CAPS BETWEEN DIFFERENT MODELS OF RECHARGEABLE HANDLES. USE OF INCORRECT BOTTOM CAP MAY DAMAGE THE BATTERY OR CAUSE THE HANDLE TO OVERHEAT. BURNS MAY OCCUR FROM AN OVERHEATED HANDLE.**

Order replacement batteries from any Welch Allyn distributor.

### Welch Allyn Convertible Handles

To interchange rechargeable battery with standard ("C") batteries:

1. Unscrew transformer module assembly.
2. Shake out rechargeable battery.
3. Insert battery converter: Product No. 710168-501 for #71000-C handle or Product No. 710168-502 for #71020-C handle (export).
4. Unscrew bottom cap and insert two ("C") batteries with plus (+) terminal first.
5. Replace bottom cap.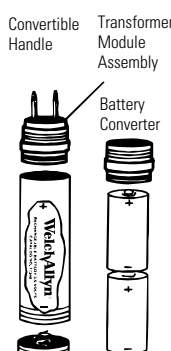

To interchange standard ("C") batteries with rechargeable battery, reverse above process.

### Special Notes

- Slight heating of handles is normal during recharge cycle.
- Batteries cannot overheat or overcharge.
- Batteries will not leak.
- Batteries will discharge gradually over a period of about 60 days when stored at room temperature (70°F). Storage at higher temperatures accelerates the discharge rate.
- For rechargeable handles that accommodate "C" or "AA" size batteries — remove batteries from handle if instrument is not in use for an extended period of time.

### Cleaning Procedure for Handles


The cleaning of all Welch Allyn power handles is easily accomplished by wiping the external surface with a damp cloth soaked with a mild detergent and water solution, or solution of 70% isopropyl alcohol and water, or a solution of 10% chlorox and water. Do not immerse. Soak the cloth in the solution and wring out to prevent excess water from entering the assembly.

Note: Excess solution entering the assembly could damage internal components. Use caution to ensure cloth is not saturated with solution.


### Battery Charger

#### Charger Instructions

1. Plug into a 110-130 V, 50-60 Hz A.C. receptacle for No. 71040, (90-110 V for 71030, 220-250 V for 71032, 71034, and 71036).
2. Connect the charging plug to charger socket on instrument or charging stand.
3. Support the instrument during charging. Do not suspend the instrument by its charging cord.
4. Batteries do not overcharge. A completely discharged battery is fully charged overnight.
5. Use the charger with either 2.5 V or 3.5 V rechargeable instruments.
6. Do not use diagnostic instruments while charging.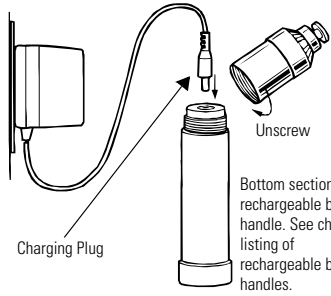

### AudioScope™ 3 with Charging Stand & CompacSet

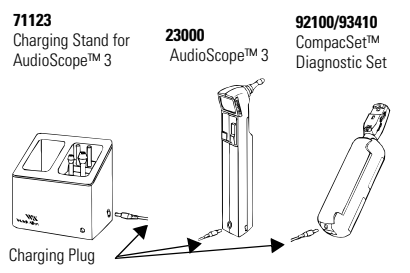
**STERILIZING/CLEANING INSTRUCTIONS:** Do NOT sterilize. Clean by wiping with a dry cloth.

**WARNING:** Not for use in the presence of flammable anesthetics.

### Charging Instructions for Rechargeable Handles for Desk Charger or Pocket Set Charger

1. To recharge, place handle in charging well, ensuring that contact is made at the base.
2. For full capacity, charge for 16 continuous hours when brand new. For subsequent recharging, place handles in well whenever not in use or when capacity is depleted.

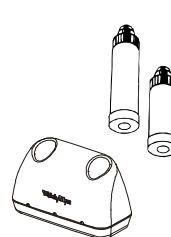### Ordering Information and Charger Specifications • Commandes et caractéristiques techniques du chargeur • Bestellinformationen und Technische Daten des Ladegeräts • Información para pedidos y especificaciones del cargador • Informazioni sull'ordinazione e dati tecnici del caricatore

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